



Lyra[®] SARS-CoV-2 ASSAY

Specimen Collection Guide

**Emergency Use
Authorization**



Study the Package Insert thoroughly before using this Procedure Card.
This is not a complete Package Insert.


Storage and Handling of Viral Transport Media

Store at 2°C to 8°C until the expiration date listed on the outer kit box.

Specimen Collection, Storage and Handling

Nasopharyngeal, nasal or oropharyngeal specimens should be collected, transported, stored, and processed according to CLSI M41-A2. Specimens should be stored at 2°C to 8°C until tested. If specimens cannot be tested within 72 hours of collection, they should be frozen at -70°C or colder until tested.

1

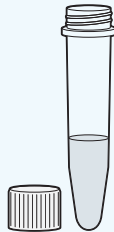

Remove Viral
Transport Media cap.

2



Insert patient specimen.
Vigorously swirl in the
media for 5 seconds.
Break the swab off into the
tube by bending the swab
shaft at a angle to break.

3



Discard the broken handle
and place into appropriate
biohazard waste container.
Cap tube and store at
2°C to 8°C until testing.